Juliette Armand

AMESON

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

EN AMESON MICROPEN

Juliette Armand's Ameson Micropen is a portable electric device of "microneedling" technology for immediate percutaneous absorption of Mesotherapy products, in sterile form, with adjustable automatic mode of pulse vibrations. The thin needles of the device penetrate the skin vertically with adjustable speed and depth conditions creating small channels/passages. The creation of a mechanical microtrauma to the epidermis caused by the thin needles adjusted on the device, activates the natural mechanism of skin cell regeneration which in synergy with the action of mesotherapy products, improves the texture and appearance of the skin. The Ameson Micropen is integrated in Juliette Armand's mesotherapy protocols with different indications for face (fine lines, deep wrinkles, acne scars, slackening, discolorations) and body (local fat, cellulite, stretch marks, deep scars, slackening, discolorations).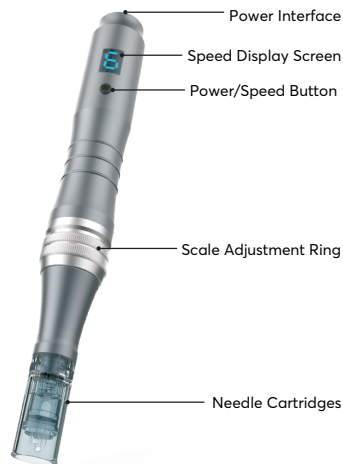

Needle Cartridges Options

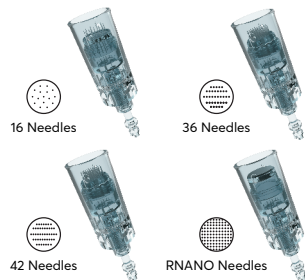Content of Packaging: The packaging contains 1 Micropen, 1 adapter, 1 USB power cord, 2 disposable tips of 36 and nano needles, while disposable tips of 16 and 42 needles can also be purchased separately. All the needles included are sterile and made of stainless steel.

General Safety Instructions: Inspect the product's integrity before use. Assemble and operate the device in accordance with the instructions indicated.

Micropen Operation: The device can operate either with the built-in battery or with a power supply. Connect the adapter to a 220V power outlet and the power cord to the Micropen socket.


Charging Status

- The power indicator stays red while charging
- The power indicator turns green after charging completed

To charge the battery, plug the adapter into a 220V socket and plug it into the power socket of the device. Unseal the sterile disposable head with the needles from the package. Apply the head to the Micropen socket, press horizontally, lock it by turning it clockwise and remove the detachable protective cap. To remove


the head, turn it counterclockwise and pull out.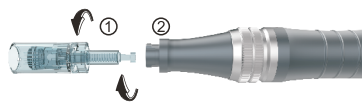

Settings: To activate or deactivate the Micropen, press the power button that is below the speed screen for 2-3 seconds.

Caution: For better protection of the device, do not place in an upright position with the disposable head at the top to prevent the cosmetic product from flowing backwards and inside the device.

In order to adjust the length of the needles, rotate the adjustment ring. By rotating it counterclockwise the length of the needles increases, while by rotating it clockwise the length of the needles decreases.

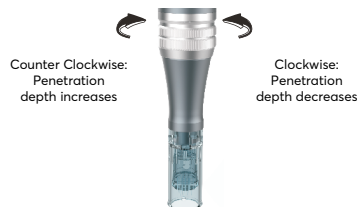The device starts at the lowest speed (1). To change the speed of pulse vibrations, press the button and select one of the 6 different default speeds.


Speed Display

Recommended Needle Length Settings

Area	Thin skin	Thick skin
Forehead	0.25mm	0.25 – 0.5mm
Glabella	0.25 - 0.5mm	0.5 – 0.75mm
Neck/Décolleté	0.25 - 0.5mm	0.5 – 0.75mm
Nose	0.25mm	0.25 – 0.5mm
Area around the eyes	0.25mm	0.25 – 0.5mm
Cheeks	0.25 - 0.5mm	0.5 – 0.75mm
Chin	0.5 - 1.0mm	0.75 – 1.5mm
Upper lip	0.25mm	0.25 – 0.5mm
Scars	1.0 – 1.25mm	1.0 – 2.0mm
Stretch Marks	1.5 – 2.0mm	1.5 – 2.5mm

Instructions for Use: Select the appropriate Juliette Armand Mesotherapy protocol to be followed and carefully study the instructions for use of the Micropen. Cleanse the skin thoroughly using the appropriate product, according to the skin type. Optionally, you may use local anaesthesia. Apply an antiseptic solution on the area where the microneedling procedure will be implemented. Apply the appropriate sterile Ameson Mesotech mesotherapy product. The application of Micropen should be performed on a layer of active substance. Activate the device and use the Micropen by dividing the face in zones. Using your free hand immobilize the skin. Start with perpendicular movements, lift periodically the Micropen and gradually cover the entire region, moving according to the following diagrams.

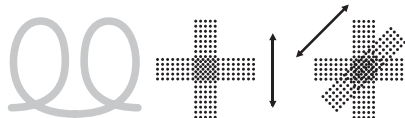Apply the appropriate Ameson Mesotherapy After

product. Complete the procedure by applying sun protection, if instructed so by the given protocol. Following the microneedling procedure, it is recommended to avoid exposure to UV, the use of sauna and steam baths, as well as the use of makeup for 24 hours.

Contraindications: Do not use on irritated and contaminated skin, in cases of acne, moles, keloids, wounds, blood coagulation disorders and problems, malignancies, fever, diabetes mellitus, autoimmune diseases, pregnancy, lactation.

Caution:

- When not in use, keep the device switched off.
- Wipe the surface of the device with a dry cloth.
- Keep the device away from direct sunlight, dust or high temperature and humidity.
- Do not allow the use, repair or replacement of the device to non-professional personnel. It may result in malfunction of the device.
- When using the device after it hasn't been used for a long period, make sure it is operating properly and safely.
- Store the device in a safe place away from children.
- Do not operate the device in static voltage other than AC 220V. This may result in fire or malfunction of the device.
- Do not touch the device or unplug the power chords with wet hands. It may result in an electric shock.
- Do not use bare cables. It may result in fire. Make sure they are replaced.
- Do not operate the device with the chord if the chord is damaged. It may cause damages to the skin.
- Unplug the device when not in use for a long time.

Side Effects: Side effects have not been documented with the proper use of the Micropen. Do not hesitate to contact us in order to express any comment on the product or the provided instructions.

Storage: Store in a dry environment, away from sunlight.

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